OSOM® Trichomonas Positive Control Kit

Catolog number 182

INTENDED USE

The OSOM® Trichomonas Positive Control Kit is intended for *in vitro* diagnostic use in quality control testing with the OSOM Trichomonas Rapid Test.

SUMMARY

The OSOM Trichomonas Positive Control Kit provides swabs coated with a dried mixture containing Trichomonas antigens. The level of Trichomonas antigen applied to the swab is sufficient to provide a positive result when used with the OSOM Trichomonas Rapid Test. Sterile rayon swabs supplied in the Trichomonas Rapid Test kit are free of Trichomonas antigens and can be used as negative controls. When used for quality control in the OSOM Trichomonas Rapid Test, these controls are useful as an aid in verifying test and operator performance.

MATERIALS PROVIDED

OSOM Trichomonas Positive Control Swabs (approximately 1 x 10⁵ cell equivalents)- 10 polyester swabs coated with an antigen mixture derived from *Trichomonas vaginalis* organisms.

MATERIALS REQUIRED BUT NOT PROVIDED

- OSOM Trichomonas Rapid Test
- Clock or Timer

WARNINGS AND PRECAUTIONS

1. DO NOT use the Controls beyond the expiration date.
2. Potential biohazard. Handle as if potentially infectious. Handle this product according to established good laboratory practices using appropriate precautions.
3. Not for patient use.

STORAGE

- The OSOM Trichomonas Positive Control Kit must be stored at room temperature (15°-30°C; 59°-86°F).
- Tightly close the canister after use to prevent deterioration of the positive control swabs.

Stability

Swabs stored in the closed canister at 15°-30°C (59°-86°F) are stable until the expiration date printed on the canister and box label.

INSTRUCTIONS FOR USE

Use the OSOM Trichomonas Positive Control Swabs as you would a patient sample in accordance with the procedure for the OSOM Trichomonas Rapid Test. Read the procedural instructions from the assay product insert prior to testing.

EXPECTED RESULTS

The Negative Control should yield a negative result as described in the Interpretation of Test Results section of the assay product insert.

The Positive Control should yield a positive result as described in the Interpretation of Test Results section of the assay product insert.

DISPOSING OF MATERIALS

Dispose of hazardous or biologically contaminated materials according to your institution’s practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state, and local requirements.

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Refer to the OSOM Trichomonas Rapid Test directional insert for additional xinformation on this assay.

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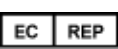
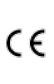
	
Sekisui Diagnostics, LLC <p>6659 Top Gun Street San Diego CA 92121 USA</p>	Sekisui Diagnostics (UK) Ltd <p>Liphook Way, Allington Maidstone, Kent, ME16 0LQ, United Kingdom</p>
	
www.sekisuidiagnostics.com <p>800 332-1042 (U.S. only)</p>	

© 2015 Sekisui Diagnostics, LLC All rights reserved

DEFINITIONS FOR SYMBOLS / SYMBOLFORKLARING / DÉFINITIONS DES SYMBOLES / DEFINITIONEN DER SYMBOLE / DEFINIZIONE DEI SIMBOLI / SYMBOLER / DEFINICIONES DE SIMBOLOS / SYMBOLDEFINITIONER / SYMBOLIEN SELITYKSET

			
Manufacturer	Batch code	Catalogue number	Temperature limitation
Producent	Lotnummer	Katalognummer	Temperaturbegrænsning
Fabricant	Code du lot	Référence du catalogue	Limites de température
Hersteller	Chargenbezeichnung	Bestellnummer	Temperaturbegrenzung
Fabbricante	Codice del lotto	Numero di catalogo	Limiti di temperatura
Tilvirket av	Partinummer	Katalognummer	Temperaturbegrænsning
Fabricante	Código de lote	Número de catálogo	Limite de temperatura
Tillverkare	Lot nummer	Katalognummer	Temperaturbegrænsning
Valmistaja	Erän numero	Luettelonumeror	Lämpötilarajoitus

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Använd före

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EC REP

Authorised representative in the EC

Repræsentant i det Europæiske Fællesskab
Mandataire dans la Communauté européenne
Bevollmächtigter in der Europäischen Gemeinschaft
Mandatario nella Comunità Europea
Autorisert representant
Representante autorizado en la Comunidad Europea
Auktoriserad representant i Europeiska Gemenskapen
Valtuutettu edustaja

© 2015 Sekisui Diagnostics, LLC All rights reserved

3086-3, 07/15