

The FastPack® IP Vitamin D Immunoassay



FAST / SIMPLE / ACCURATE

The FastPack® IP Vitamin D Immunoassay

The FastPack® IP Vitamin D Immunoassay, for use on the FastPack® IP System, allows any testing facility to produce point-of-care results in 12 minutes with less than one minute of hands-on time.

- Enhanced quality of service for patients
- Ability to monitor and treat in one visit
- Improved patient compliance and outcome
- Laboratory accuracy and precision



SEKISUI
DIAGNOSTICS

Because every result matters™

The FastPack® IP Vitamin D Immunoassay

Simple Process

PRETREAT SAMPLE

- 1) Pipette 100 µL of sample into prefilled vial
- 2) Recap and invert 3 times



Dispense sample into the FastPack® Injection Port



Place FastPack® into the FastPack® IP Instrument



Press the START button

The FastPack® IP Vitamin D Assay Specifications:

- Quantitative measurement of 25-hydroxyvitamin D
- Lower Limit of 12.9 ng/mL
- Upper Limit of 150 ng/mL
- Assay time: 12 minutes
- CPT CODE: 82306

ORDERING INFORMATION

CONFIGURATION

CATALOG NUMBER

FastPack® IP Vitamin D
(Includes Calibrator, Controls, and Pre-Treatment Buffer)

30 Tests

25000068

FastPack® IP Vitamin D Verification

Kit

25000064

For In Vitro Diagnostic Use

THE AMERICAS

Sekisui Diagnostics, LLC
4 Hartwell Place
Lexington, MA 02421
Phone: 800 332 1042
Fax: 800 762 6311
Email: questions@sekisui-dx.com

INTERNATIONAL

Sekisui Diagnostics (UK) Limited
Liphook Way, Allington
Maidstone, Kent ME16 0LQ, UK
Phone: +44 1622 607800
Fax: +44 1622 607801
Email: info@sekisui-dx.com

www.sekisuidiagnostics.com

SEKISUI
DIAGNOSTICS

Because every result matters™