

HELPING YOU HIT THE MARK IN FLU TESTING



The **Acucy™ Influenza A&B Test** on the Acucy™ System provides clinicians flexibility in workflow and accurate, standardized results for improved patient care.

ACCURATE

- ✓ Innovative Composite Comparator with molecular and culture
- ✓ Best-In-Class Flu A performance*
- ✓ Definitive and standardized results

SIMPLE


- ✓ Read Now or Walk Away mode
- ✓ Results within 15 minutes
- ✓ Nasal & Nasopharyngeal swab types

*When compared to immunoassay products in market (refer to manufacturer instructions for use)



Accurate Performance

Composite Comparator: Nasal and Nasopharyngeal Swabs

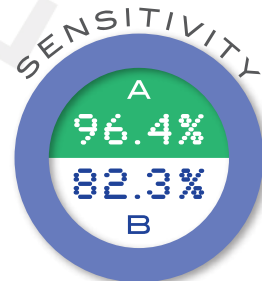


Influenza A - 96.4% (95% CI: 93.1% - 98.2%)
Influenza B - 82.3% (95% CI: 75.6% - 87.4%)

Influenza A - 96.0% (95% CI: 94.4% - 97.2%)
Influenza B - 98.1% (95% CI: 96.9% - 98.8%)

Test Procedure

EXTRACT PATIENT SPECIMEN

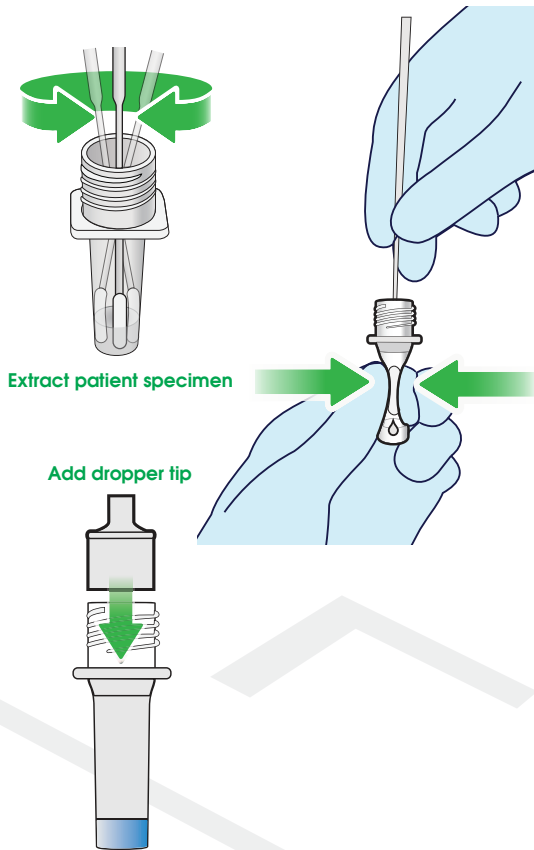

RUN TEST

WALK AWAY/NORMAL MODE:


Reader will time test and display results

- ✓ Remove cassette from foil pouch
- ✓ Place cassette in Reader
- ✓ Add sample to cassette
- ✓ Close reader drawer
- ✓ Reader will time test for 15 minutes
- ✓ Reader displays results

READ NOW MODE:

Test timed outside of the Reader

- ✓ Remove cassette from foil pouch
- ✓ Add sample to cassette
- ✓ Set timer for 15 minutes
- ✓ Place cassette into Reader
- ✓ Close Reader drawer
- ✓ Reader displays results

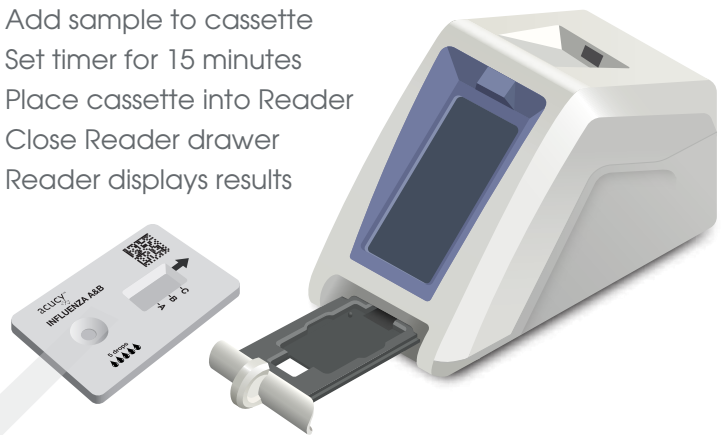

DESCRIPTION	CAT#	QTY	CPT CODE
Influenza A&B Test Kit 25 tests; 2 extra tests for QC	1010	25 Tests	87804QW, 87804QW-59
Influenza A&B Control Set	1011	10 Swabs	
System—Reader & Printer	1039	1	
Exclusive USB Memory Drive	1021	1	
Calibration Device	1031	1	
Printer Roll	1019	1	



THE AMERICAS

Sekisui Diagnostics, LLC
One Wall Street
Burlington, MA 01803
Phone: 800 332 1042
Fax: 800 762 6311
Email: questions@sekisui-dx.com

INTERNATIONAL

Sekisui Diagnostics (UK) Limited
Liphook Way, Allington
Maidstone, Kent, ME16 0LQ, UK
Phone: +44 1622 607800
Fax: +44 1622 607801
Email: info@sekisui-dx.com

SEKISUI
DIAGNOSTICS

Because every result matters™

www.sekisuidiagnostics.com