

ACUCY™ INFLUENZA A&B TEST External Quality Control (QC) Quick Reference Guide

instructions for performing the test. 42 USC263a(c).



Because every result matters™

REF

1010





External QC requirements should be established in accordance with local, state, and federal regulations or accreditation requirements. Minimally, Sekisui Diagnostics recommends that External Quality Control be run with each new lot, shipment received, and with each new operator.

Facilities with a Certificate of Waiver may perform this test and must follow the manufacturer's

IMPORTANT: Read and follow the complete external QC test procedure before performing the test. Refer to the Instructions for Use (IFU) and System Manual for more complete information.

A. RUN QC TEST (A+/B- CONTROL SWAB)


Running external control requires running both positive control samples (A+/B- & A-/B+) requiring a total of 30 minutes. Dropped cassettes (regardless of whether samples were added to the cassettes) should be discarded.





- 1- Select QC TEST from MAIN MENU
- 2- Select OPERATOR ID
- 3- Select test FLU A&B



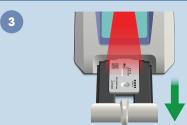

Remove cassette from foil pouch. Open drawer slowly until it clicks into place. Insert cassette with arrow facing Reader. Reader will beep when cassette has been detected. Test cassettes must be used within 30 minutes of opening or will need to be discarded.

B. EXTRACT CONTROL SWAB



Insert swab into extraction buffer vial and mix against side of vial 10 times.





Remove swab while squeezing middle of vial to remove liquid from swab.



Attach dropper tip to vial. Press tightly to seal.



C. RUN QC TEST





Gently mix sample. Hold vial vertically and gently add 5 drops to sample well. DO NOT add sample to the results window.

8





Immediately close drawer tightly. Reader will automatically time the 15-minute development.

9



When Flu A+/B- is complete, open drawer, remove cassette and press NEXT.

E. QC RESULTS

D. RUN QC TEST (A-/B+ CONTROL SWAB)




When prompted by Reader, scan barcode on Positive A-/B+ Control swab pouch. (BLUE LABEL). Reader will beep when swab pouch barcode has been detected.







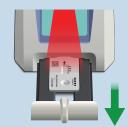


Remove cassette from foil pouch.
Insert cassette with arrow facing
Reader. Reader will beep when
cassette has been detected.
Test cassette must be used within

1est cassette must be used within 30 minutes of opening or will need to be discarded.



Follow Steps 4-8 to Extract Control Swab and Run QC Test with the Influenza A-/B+ Control Swab.



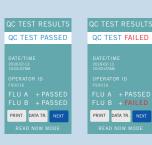

QC TEST results will be displayed as PASSED or FAILED.

If External QC testing fails, repeat the test using new control swabs, reagents and cassettes.

TECHNICAL SUPPORT

If the QC test does not perform as expected, contact Sekisui Diagnostics Technical Support at 800-332-1042 (U.S. Only)