EN

## MICROALBUMIN MULTI-CALIBRATOR SET

CATALOGUE NUMBER: SE-252

SIZE: 6 x 1 mL

### INTENDED USE

For IN VITRO diagnostic use as a calibrator for the Sekisui Diagnostics' Microalbumin Assay for quantitation of albumin in human urine.

### CHARACTERISTICS

The calibrators included are human albumin standards containing known quantities of human albumin. There is also a calibrator containing only diluent (150 mmol/L sodium chloride). These calibrators are to be used with Sekisui Diagnostics' Microalbumin Assay.

### WARNINGS & PRECAUTIONS FOR USE

**IVD**

For *In Vitro* Diagnostic Use.

**Rx ONLY**

Avoid contact with skin and eyes. Avoid ingestion. See Safety Data Sheet for additional information.

Exercise the normal precautions required for handling all laboratory reagents. This product has been prepared exclusively from the urine of donors tested individually and shown by FDA-approved methods to be free from HBsAg and antibodies to HCV and HIV. However, as no test method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient sample. In the event of exposure, the directives of the responsible Health Authorities should be followed.

### PREPARATION, STORAGE & STABILITY

Supplied calibrators are ready for use.

The unopened calibrators are stable at 2-8°C until expiry date.

When opened, calibrators are stable at 2-8°C for 30 days. Replace cap and return calibrators to 2-8°C promptly after use.

Calibrators should be brought to room temperature (18-26°C) before dispensing. Mix gently before use.

### CALIBRATOR DETERIORATION

The calibrator solutions should be clear. Turbidity would indicate deterioration.

### DISPOSAL

Calibrators must be disposed of in accordance with all Federal, Provincial, State and local regulations.

## PERFORMANCE CHARACTERISTICS

NOTE: ASSIGNED VALUES ARE LOT SPECIFIC. Do not interchange lots of calibrator with inserts of a different lot. Please refer to the next page for value assignment.

### PRECISION STUDIES

Precision information is available upon request from Sekisui Diagnostics.

### ASSIGNED VALUES (MICROALBUMIN)

SEKISUI DIAGNOSTICS' MICROALBUMIN REAGENT Cat. No. 252-20		
CALIBRATOR	ASSIGNED ALBUMIN VALUE	
	mg/L	mg/dL
Level 1	0.0	0.00
Level 2	5.0	0.50
Level 3	10.0	1.00
Level 4	50.0	5.00
Level 5	100.0	10.00
Level 6	300.0	30.00

All trademarks, brands, product names and trade names are the property of their respective companies.

Manufactured by:

#### The Americas

Sekisui Diagnostics P.E.I. Inc.  
70 Watts Avenue  
Charlottetown, PE C1E 2B9  
Canada

Phone: 800-565-0265

Fax: 902-628-6504

Email: [questions@sekisuidiagnostics.com](mailto:questions@sekisuidiagnostics.com)

[peidiagnosticttechnical@sekisuidiagnostics.com](mailto:peidiagnosticttechnical@sekisuidiagnostics.com)

[www.sekisuidiagnostics.com](http://www.sekisuidiagnostics.com)

#### International

Sekisui Diagnostics (UK) Limited  
Liphook Way  
Allington, Maidstone  
KENT, ME16 0LQ, UK

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The word SEKURE and the Sekure logo are registered trademarks of Sekisui Diagnostics, LLC.
MDSS GmbH