



POINT- OF-CARE



OSOM® RSV/Adeno Test

FOR THE QUALITATIVE DETECTION OF RESPIRATORY SYNCYTIAL VIRUS (RSV) AND/OR ADENOVIRUS ANTIGENS

Features and Benefits

- **Accurate** – High Sensitivity. High Specificity
- **No Need for Instrument** – Can be performed anywhere
- **Fast and Simple Workflow** – Less than 1 minute hands-on time. Results in 10 minutes
- **Differentiate between RSV and Adeno** – Easy-to-read 3 distinct lines
- **Exceptional Customer Service** – Online training and certification by certified medical technologists

Performance Characteristics

	Sensitivity*	Specificity**	Sample Type	Storage Temperature	Time to Result
RSV	98.9%	98.6%	Nasal Swab, Throat Swab or Nasal Suction Fluid	15-30°C	10 minutes
Adeno	91.9%	97.4%			

*Compared to In Vitro Diagnostic Agent (Immunochromatography)

**Throat swab only for detection of Adenovirus. Refer to IFU for additional information



SENSITIVITY



SPECIFICITY




OSOM[®] RSV/Adeno Test




SIMPLE TO USE / RESULTS IN MINUTES

1 ADD 0.3 ML EXTRACTION BUFFER TO TEST TUBE

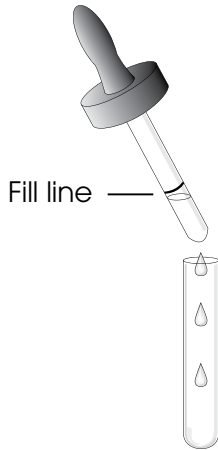


2 MIX SWAB IN BUFFER


10X

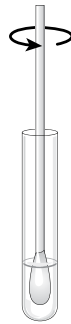

3 SQUEEZE OUT AS MUCH LIQUID AS POSSIBLE FROM SWAB

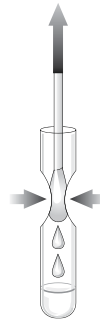


4 INSERT TEST STICK AND READ RESULTS AT 10 MINUTES

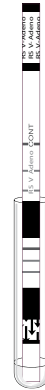


INTERPRETATION OF RESULTS


RSV POSITIVE

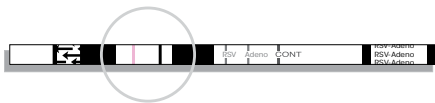

BOTH RSV AND ADENOVIRUS POSITIVE

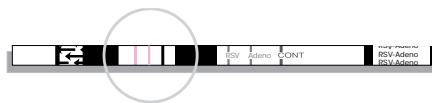


ADENOVIRUS POSITIVE

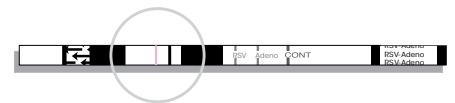

NEGATIVE



INVALID

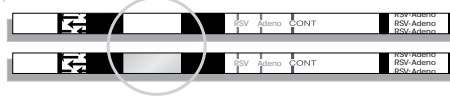

Ordering Information

	Configuration	Catalog Number
OSOM RSV/ADENO TEST	25 tests per kit	197E
OSOM RSV/ADENO CONTROLS	5-RSV Positive Swabs & 5-ADENO Positive Swabs	198E

To learn more about all of our OSOM Tests scan the code or go to osomtests.com



INTERNATIONAL

SEKISUI Diagnostics (UK)
Limited Liphook Way, Allington
Maidstone, Kent, ME16 0LQ, UK
Phone: +44 1622 607800
Fax: +44 1622 607801
Email: info@sekisui-dx.com



SEKISUI
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com