OSOM® RSV/Adeno Control Kit

Catolog number 198E

INTENDED USE

The OSOM® RSV/Adeno Control Kit is intended for in vitro diagnostic use in quality control testing with the OSOM RSV/Adeno Test.

SUMMARY

The OSOM RSV/Adeno Control Kit provides swabs coated with a dried mixture containing RSV or Adenovirus antigens. The level of RSV or Adenovirus antigen applied to the swabs is sufficient to provide a positive result when used with the OSOM RSV/Adeno Test. The RSV control swab acts as a negative control for the Adenovirus antigen and conversely, the Adenovirus antigen acts as a negative control for the RSV antigen. When used for quality control in the OSOM RSV/Adeno Test, these controls are useful as an aid in verifying test and operator performance.

MATERIALS PROVIDED

- 5 OSOM RSV Control Swabs (blue shaft foam swab packaged with a desiccant tablet)
 - Formalin inactivated RSV containing 0.05% sodium azide. Inactivity confirmed by inability of virus to infect cell culture.
 - Result is representative of a mid-level positive.
- 5 OSOM Adenovirus Control Swabs (pink shaft foam swab packaged with a desiccant tablet)
 - Formalin inactivated Adenovirus containing 0.05% sodium azide. Inactivity confirmed by inability of virus to infect cell culture.
 - Result is representative of a mid-level positive.

MATERIALS REQUIRED BUT NOT PROVIDED

- OSOM RSV/Adeno Test
- Clock or Timer

WARNINGS AND PRECAUTIONS

- DO NOT use the Controls beyond the expiration date.
- Potential biohazard. Handle as if potentially infectious. Handle this product according to established good laboratory practices using appropriate precautions.
- Not for patient use.

STORAGE

- The OSOM RSV/Adeno Control Kit must be stored at room temperature (15°-30°C; 59°- 86°F).

Stability

Swabs stored in the closed canister at 15°-30°C (59°- 86°F) are stable until the expiration date printed on the canister and box label.

INSTRUCTIONS FOR USE

Use the OSOM RSV/Adeno Control Swabs as you would a patient sample in accordance with the procedure for the OSOM RSV/Adeno Test. Read the procedural instructions from the assay directional insert prior to testing.

EXPECTED RESULTS

The RSV Control swab should yield a positive RSV and a negative Adenovirus result.

The Adenovirus Control swab should yield a positive Adenovirus and a negative RSV result. Refer to the assay directional insert for a full description in the Interpretation of Test Results section.

DISPOSING OF MATERIALS

Dispose of hazardous or biologically contaminated materials according to your institution's practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state, and local requirements

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© 2019 Sekisui Diagnostics, LLC All rights reserved