



POINT- OF-CARE



OSOM® H. pylori Test

FOR THE DETECTION OF Anti-H. pylori IgG ANTIBODIES

Features and Benefits

- **Accurate** – High Sensitivity/Specificity
- **Fast** – and easy work flow
- **CLIA Waived** – For finger stick whole blood
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

Performance Characteristics

Sensitivity/Specificity	Sample Type	Storage Temperature	Time to Result
95.9%/89.1%*	Serum, plasma, whole blood**	2-30 °C/36-86°F	10 minutes

*Compared to biopsy/histology

**CLIA Waived for finger stick whole blood




OSOM® H. pylori Test




Henry P. Lori

SIMPLE TO USE / RESULTS IN MINUTES


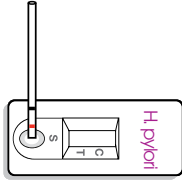


1 FOR SERUM OR PLASMA, FILL CAPILLARY TUBE TO THE RED LINE. FOR WHOLE BLOOD, FILL TO THE BLACK LINE. ADD SAMPLE BY LIGHTLY TAPPING CAPILLARY TUBE ON UPPER AREA OF THE SAMPLE WELL.

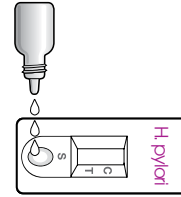


2 ADD 3 DROPS OF DEVELOPER TO LOWER AREA OF SAMPLE WELL

3 READ RESULTS AT 10 MINUTES

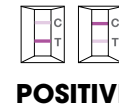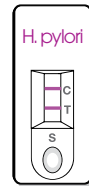

POSITIVE

NEGATIVE

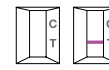

INVALID

Ordering Information

Description	Configuration	Catalog Number	Storage	CPT Code
OSOM H. PYLORI TEST	25 Tests	175	Room Temp	86318QW (Whole Blood)
H. PYLORI CONTROL KIT	2 X 1mL vials (Pos & Neg)	176	Refrigerated	86677 (Serum, Plasma)

To learn more about all of our OSOM Tests scan the QR code or go to osomtests.com



THE AMERICAS

SEKISUI Diagnostics, LLC
One Wall Street
Burlington, MA 01803
Phone: 800 332 1042
Fax: 800 762 6311
Email: questions@sekisui-dx.com

SEKISUI
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com

