EN

TPLA Calibrator Set

INTENDED USE

For the calibration of the TPLA assay. The TPLA Calibrator Set can only be used with the Sekisui Diagnostics TPLA Reagent.

SUMMARY AND PRINCIPLE

See the TPLA Reagent package insert.

REAGENTS/COMPOSITION

The TPLA Calibrator Set contains 5 calibrators based on human serum and bovine serum albumin (BSA) with assigned concentrations of Anti-*Treponema pallidum* antibodies.

Precautions and Warnings

1. For In Vitro Diagnostic Use.
2. Do not use the calibrators beyond the expiration date printed on the labels.
3. **Warning:** Human source material. Treat as potentially infectious. Each plasma donor unit used in the preparation of this product has been tested by an FDA-approved method and found non-reactive for HBsAg, and antibodies to HCV and HIV-1. Because no known test method can offer complete assurance that Hepatitis B virus, Human Immunodeficiency Virus (HIV) or other infectious agents are absent, all human-based products should be handled in accordance with good laboratory practices using appropriate precautions.¹
4. **Caution:** TPLA Calibrators contain 0.1% sodium azide as a preservative. Sodium azide may react with lead and copper plumbing to form potentially explosive metal azide build up. Flush with copious amounts of water when discarding material.
5. Disposal of all waste material should be in accordance with local guidelines.
6. **Warning:** This product contains syphilis positive plasma and should be handled as a potentially bio hazardous material in accordance with universal/ standard precautions.

Handling

Calibrator bottles 1-5 are ready for use. Mix carefully before use. Avoid the formation of foam.

Storage

Store at 2 - 8°C. Do not freeze.

Stability

Unopened: Up to stated expiration date at 2 - 8°C.
After opening: 8 weeks at 2 - 8°C.
Store calibrator tightly capped when not in use.

Indications of Deterioration

Presence of turbidity or microbial growth may indicate deterioration.

PROCEDURE

Materials Provided

Description	Configuration	Catalog Number
TPLA Calibrator Set	5 levels x 2mL vials	515132

Materials Required but not Provided

Description	Configuration	Catalog Number
TPLA Reagent 1	1 x 60mL	486647
TPLA Reagent 2	1 x 10mL	
TPLA Control Set	Level A 1 x 3mL Level B 1 x 3mL	515149

- Volumetric pipette or calibrated single channel pipette.
- Distilled/deionized/Type II water or equivalent.²
- Analyzer capable of running two-reagent chemistries.

Calibration

When using this calibrator set, treat it in exactly the same manner as a patient sample.

The TPLA assay requires a multipoint calibration curve. The values of the TPLA calibrators, which can be found on the calibrator vial labels, are traceable to an in-house standard.

Refer to the instrument operator's manual for analyzer specific calibration procedures and for guidance in determining calibration frequency.

Quality Control values should be within the expected ranges.

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1. Richardson JH and Barkley WE, eds. Biosafety in Microbiological and Biomedical Laboratories, U.S. Dept. of Health and Human Services, Public Health Service, HHS Publication No. (CDC) 84-8395, Washington, DC: 1984.
2. Clinical and Laboratory Standards Institute. Preparation and Testing of Reagent Water in the Clinical Laboratory, 3rd Edition. Approved Guideline (Genehmigte Richtlinien). CLSI Document C03-A3. Wayne, PA: 1997.
3. Data on file at SEKISUI.

Definitions for Symbols/ Definition Der Symbole/ Définition des symboles/ Definizioni dei simboli/ Definiciones de los símbolos/ Definice symbolů/ Определение символов

REF

Catalog number
Bestellnummer
Référence
Numero di catalogo
Número de catálogo
Katalogové číslo
Номер по каталогу

Temperature limitation
Temperaturbegrenzung
Limites de température
Limite di temperature
Limite temperature
Teplotní omezení
Температурное ограничение

IVD

For in vitro diagnostic use
Für die In-Vitro-Diagnostik bestimmt
Pour diagnostic in vitro
Per uso diagnostico in vitro
Para uso en diagnóstico in vitro
K diagnostickému použití in vitro
Только для in vitro диагностики

Manufactured by
Hergestellt von
Fabriqué par
Prodotto da
Fabricante
Výrobce
Изготовлено

Use by
Zu verwenden bis
Utiliser avant
Usare entro il
Fecha de caducidad
Datum spotřeby
Использовать до

Consult instructions for use
Gebrauchsanweisung beachten
Consulter le mode d'emploi
Consultare le istruzioni per l'uso
Consulte las instrucciones de uso
Prostudujte si návod k použití
Обратитесь к инструкции по применению

LOT

Batch code
Chargenkennung
Code de lot
Codice del lotto
Código del lote
Kód šarže
Код партии

Caution, consult accompanying documents
Achtung, Begleitunterlagen beachten
Attention, consulter la documentation jointe
Attenzione, consultare i documenti di accompagnamento
Precaución, consulte los documentos anexos
Upozornění, prostudujte si příloženou dokumentaci
Внимание! Обратитесь к сопровождающей документации

EC REP

Authorized Representative in the European Community
Autorisierte Vertretung in der Europäischen Gemeinschaft
Représentant agréé dans la Communauté européenne
Rappresentante autorizzato nell'Unione Europea
Representante autorizado en la Comunidad Europea
Autorizovaný zástupce pro Evropské společenství
Уполномоченный представитель в Европейском сообществе

Biological risks
Biologisches Risiko
Risques biologiques
Rischi biologici
Riesgos biológicos
Biologická rizika
Биологический риск

SEKISUI MEDICAL CO., LTD.
1-3 , Nihonbashi 2 -chome, Chuo-ku
Tokyo, 103-0027, Japan

Medical Device Safety Service GmbH (MDSS)
Schiffgraben 41
30175 Hannover, Germany

April 2019 KI515132. 03

The word SEKURE and the Sekure logo are registered trademarks of Sekisui Diagnostics, LLC.

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Das Wort SEKURE und das SEKURE-Logo sind Marken-zeichen von Sekisui Diagnostics, LLC.
Le terme SEKURE et le logo Sekure sont des marques déposées de Sekisui Diagnostics, LLC.
La parola SEKURE e il logo Sekure sono marchi commerciali registrati di Sekisui Diagnostics, LLC.
La palabra SEKURE y el logotipo de SEKURE son marcas comerciales registradas de Sekisui Diagnostics, LLC.
Slovo SEKURE a logo Sekure jsou registrované ochranné známky společnosti Sekisui Diagnostics, LLC.