

Acetaminophen Transition Support – Calibrator Information

The Sekisui Diagnostics manufactured **Acetaminophen PN 506** is available as a replacement for the currently unavailable Acetaminophen assay, Abbott List Number 2K99-20 (Sekisui PN 505).

The 506 assay comes complete with an Acetaminophen Calibrator. The Calibrator included with **Acetaminophen PN 506** is equivalent to the Acetaminophen Calibrator, Abbott List Number 2K99-01.

Calibration stability of the assay is 24 hours and is included in the assay parameters. It is recommended that customers dispense only 3 drops of calibrator into the sample cups. Separate calibrator kits are not available.

Internal reference standards are created for Acetaminophen using a USP grade reference Acetaminophen material (not less than 98% and not more than 101% of paracetamol on an anhydrous basis). Acetaminophen calibrator is manufactured gravimetrically and tested against these internal reference standards.

If you have questions regarding your options, or to order this replacement assay, U.S. Customers please contact Abbott Customer Service at 1-877-4ABBOTT. Customers outside the U.S., please contact your local area Customer Service.

If you require assistance with the replacement product or support tools, please contact Sekisui Diagnostics Technical Support at 1-800-565-0265 or via email at PEIDiagnosticsTechnicalDL@sekisui-dx.com.