



POINT-OF-CARE



aptitude
Metrix[®]
COVID-19 Test

Provided by
SEKISUI
DIAGNOSTICS

Metrix[®] COVID-19 Test

A MOLECULAR LAB ANYWHERE AND EVERYWHERE

Features and Benefits

- **Molecular-Accuracy without the wait** - Improve clinical outcomes by administering immediate treatment in just 30 mins.
- **Accessible molecular testing** - Breakthrough platform designed to eliminate cost-barriers
- **Reduce repeat testing** - No need for serial or confirmation testing*
- **Alleviates testing restrictions** - Indicated for symptomatic and asymptomatic testing allowing for screening prior to medical procedures, before/after travel or in work/school programs
- **Simple workflow** - Can be performed in any point-of-care setting
- **Convenient sampling** - The only test with dual-sample types (nasal swab or saliva) providing you with flexibility for the best patient experience
- **Exceptional Support** - Experienced support teams and online training modules to help streamline your testing implementation

Performance Characteristics

No serial testing required. No confirmatory testing needed*. A result you can trust the first time.

Sample Type	Positive Percent Agreement** (PPA)	Negative Percent Agreement** (NPA)	Limit of Detection***	Storage Temperature
Nasal Swab	97%	99%	2000 GE/Swab (667 GE/uL 3mL VTM equivalent)	15-30°C/59-86°F
Saliva	90%	99%	20 GE/uL saliva sample	15-30°C/59-86°F

*Nasal samples only. **Rounded numbers compared to RT-PCR. See IFU. ***Refer to IFU for additional LOD information.



PPA
NPA

NASAL
97%
99%

SALIVA
90%
99%

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA; This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb 3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

Metrix® COVID-19 Test


Experience THE 3-S RULE to sample processing: **SAMPLE, SNAP, SLIDE**



1. SAMPLE

Collect a nasal swab sample for the best accuracy.

Saliva can also be used by depositing into the collector.

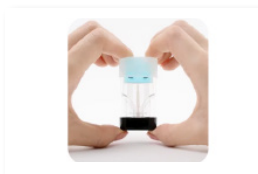

2. SNAP

Insert the swab into the collector, snap off and discard the swab handle.

Snap the cap onto the collector and then onto the sensor.



3. SLIDE

Slide the sensor onto the reader.

Read result within 30 minutes or less.

Interpretation of Results

Multicolor test status indicators with direct printed labels, removes human error and judgment.

Positive



Negative



Invalid



Ordering Information

Test Description	Configuration	Catalog Number	CPT Code
Metrix Reader	1 Reader	MTRX-RDR	
Metrix COVID-19 Test	25 Tests	MTRX-C19-25PK	87635QW
Metrix Molecular Bundle	1 Reader + 50 Tests	MTRX-C19-B1	87635QW
Microbix COVID-19 Controls (for use with Metrix COVID-19 Test)	1 SARS-CoV-2 Positive Swab, 1 SARS-CoV-2 Negative Swab	RED-S-19/99-P-01	

To learn more about the Metrix Test,
scan the code or go to metrixmolecular.com



Provided by
SEKISUI Diagnostics, LLC
One Wall Street
Burlington, MA 01803
Phone: 800 332 1042
Email: questions@sekisui-dx.com
sekisuidiagnostics.com

Designed and manufactured by
Aptitude Medical Systems Inc.
125 Cremona Drive, Suite 100
Goleta, CA 93117



80-8958-00-02 04/24