



# Metrix™ Reader

Quick Start Guide

IVD

For use with the Metrix COVID-19 Test kit (available separately).  
For use under Emergency Use Authorization (EUA) only.

## Start Here

### 01 Power Up


Connect reader to power supply.  
The center light will turn solid  
(not flashing) when ready.

### 02 Collect Sample

Open the Metrix COVID-19 Test kit (available separately) if you  
have not done so already.

The Metrix COVID-19 Test kit will guide you through how to  
collect and run your sample.

Scan QR code for  
interactive instructions  
and video demonstration.

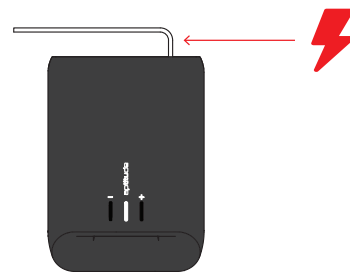

## Reader Statuses



Indicates  
flashing light



### Starting Up

The reader is starting up. Wait  
until the center light is solid  
white before inserting a sensor.



### Positive Result

The test is complete and  
SARS-CoV-2 was detected in  
the sample.



### Ready

The reader is ready to start a  
test.



### Negative Result

The test is complete and  
SARS-CoV-2 was NOT  
detected in the sample.



### Test Running

The reader is running a test.  
DO NOT remove the sensor or  
unplug the reader.



### Invalid Result

The test is complete and the  
result is invalid. Repeat with a  
new Metrix COVID-19 Test kit.

## Troubleshooting



### Test Error

Remove sensor and firmly press down on collector. Firmly reinsert sensor into reader. If error persists, discard sensor and use a new test kit.



### Canceled Test

The test did not complete. Discard the sensor and run the test with a new test kit.



### Hardware Failure

There is an error with the reader. Disconnect and reconnect the power.



Indicates flashing light



### No Power

Check all electrical connections. The reader is not receiving power.

If troubleshooting fails to resolve any problem persist please contact support.

If your Metrix Reader needs to be disposed of, please place in electronic waste.

For support, please contact us at:

1.888.934.2253  
support@aptitudemetrix.com

Or visit: AptitudeMetrix.com.

## Legend of Symbols



For vitro diagnostic use



Direct current (DC) voltage



Temperature limitations of the product



Keep dry



Please consult the instruction manual



Dispose of in electronic waste



Do not use if packaging is damaged



Manufacturer of device



Date of manufacture



Manufacturer's catalog number



Certification that the electromagnetic interference from the device is under limits approved by the Federal Communications Commission.

## Warnings and Precautions

- Do not use components that are visibly damaged.
- The Metrix Reader can be cleaned by wiping the exterior with disinfectant. Do not spray disinfectant into or onto the reader.
- If a power failure occurs or if the Metrix Reader is unplugged while the Sensor is inserted, the test result is invalidated. The test should be redone with a new test kit.
- Use only the provided power cable and power adapter.
- This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

For in vitro diagnostic use.

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA.

This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

The Metrix COVID-19 Test and Metrix Reader are for FDA Emergency Use Authorization (EUA) only



Aptitude, Metrix, and the Aptitude logo are trademarks of Aptitude Medical Systems, Inc. All Rights Reserved.